



IMPLANTS DIFFUSION
INTERNATIONAL

**Recommendations for sterilization and cleaning of surgical drill sets
(for MANUAL CLEANING ONLY)**

- 1) Use **powder-free** gloves (also for the assistant)
- 2) Soak the surgical set with the instruments in a decontamination tank during 15 min.
- 3) Remove all the instruments from the set.
- 4) Clean each instrument with a brass brush (**brass only**) in order to remove any bone fragment.
- 5) Put the instruments + the base plate in an ultrasonic bath containing a decontamination product for 15 minutes.
- 6) Change the gloves (powder-free).
- 7) Rinse both instruments and base plate in clean water during 5 minutes. Renew water.
Repeat the process 5 times 5 minutes.
- 8) Dry all parts.
- 9) Put the instruments back on the base plate.
- 10) Sterilize by autoclaving at 134°C 18 minutes according to the instructions of European Pharmacopoeia (8th Edition)

NB : Ignoring this protocol can result in the failure of the surgery and in the loss of the medical device.

NB2 : We strongly recommend not to use automatic washing machines.