



IMPLANTS DIFFUSION
INTERNATIONAL

Drilling recommendations

Maxilla :

- 1) 1st drilling \varnothing 2 mm with irrigation 650 rpm.
- 2) 2nd and 3rd drilling without irrigation at 150 rpm. With irrigation at 650 rpm (with the RBS® drill) or at 1500-2000 rpm (with the 4 bladed drill).
- 3) Rinse abundantly the socket with tobramycin, diluted with physiological serum before implant setting.

Mandible :

- 1) 1st drilling \varnothing 2 mm with irrigation 650 rpm.
- 2) 2nd and 3rd drilling and following with irrigation at 650 rpm.
- 3) Rinse abundantly the socket with tobramycin, diluted with physiological serum before implant setting.

Implant setting :

- 1) Place the implant at 1.5 mm subcrestally whatever the implant line.
- 2) Screw the closing and healing caps at 5N.cm by hand.

Expert advice : soak the implant, the closing cap and the healing cap in the tobramycin (or gentamicin) antibiotic (75 mg) diluted in 20 cl of physiological serum before placing it.